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Starter-Set zum eSense EEGenius

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Instructions for Use

eSense EEGenius Starter Set

Please read completely before first use and keep for future reference.

1. Intended use

The starter set contains the accessories needed for neurofeedback training with the eSense EEGenius: the head cap, the electrodes, the felt pads and the electrolyte solution. These parts establish the electrical contact between the scalp and the measuring device.

The set is intended for **training at home**, used by adults on themselves. It is not intended for clinical use, not for use by third parties on several people, and not for monitoring vital functions.

The set and the eSense EEGenius are not medical devices. They are not intended to diagnose, prevent, monitor, treat or alleviate disease. Do not use the training as a substitute for medical or therapeutic treatment.

2. Contents

Part	Qty	needed per session
Head cap	1	1
Head electrodes	2	1 — the second is a spare
Ear electrodes	3	2 — the third is a spare
Rubber bands for the ear clips	5	2
Felt pads	15	3 per session
Electrolyte solution NaCl 0.9 %, 50 ml	1 bottle	—

A session therefore requires **one head electrode and two ear electrodes**. The remaining electrodes are spares so that a damaged electrode does not interrupt your training.

3. Use

- Insert a felt pad into each electrode you need.
- Moisten the felt pad with the electrolyte solution until it is **evenly moist but not dripping**.
- Put on the head cap and place the head electrode in the position described in the device manual. The cap should rest on the head, not press.
- Attach the ear electrodes to the earlobes using the rubber bands.
- Connect the electrodes to the sensor and start the training.

The most common mistake is using too much solution. If liquid runs over the edge of an electrode, it forms a conductive bridge between the front and the back. The software then shows negative impedance values and the measurement stops. When in doubt, use less.

Do not let the pipette tip touch skin, hair or felt, and close the bottle immediately after use.

4. Cleaning, care and service life

Electrodes. Remove the felt pads after every use. Never let them dry inside the electrodes — this damages the coating. Store the electrodes dry. Rinse salt deposits with lukewarm water and let them dry completely. Do not sterilise, boil or machine-wash.

Electrode service life. Based on our experience from several hundred applications, around **100 sessions** per electrode are achievable. The electrodes are not single-use items, but they are not permanent components either: the silver chloride contact surface wears with every application. Replace an electrode as soon as the coating is visibly detached or damaged, or if impedance values remain poor despite a fresh felt pad and correct dosing.

Felt pads. Rinse with clear water every two to three uses to prevent salt build-up.

Replace after 15 training sessions at the latest, sooner if a pad is soiled, discoloured or frayed.

Head cap. Hand-wash with lukewarm water and hand soap, air-dry. Not for the washing machine or tumble dryer, do not iron, do not dry-clean.

All parts in contact with skin are intended for use by **one person**. Do not share them.

5. Warnings

For external use on intact scalp only.

- Do not use on broken, inflamed or irritated skin.
- Do not apply the electrolyte solution to eyes, wounds, mucous membranes or body orifices; do not swallow, inject or inhale it.

- Stop using the set if redness, itching or other irritation occurs and does not subside after the session.
- Keep out of reach of children. Not suitable for children under three; the rubber bands and small parts can be swallowed.
- Do not use while the sensor is charging, while driving or while operating machinery.
- Do not modify any of the parts.

If you have a pacemaker, an implant, a disorder of the nervous system or epilepsy, consult your doctor before use.

6. Shelf life and storage

Store the **electrolyte solution** cool, dry, dark and tightly closed. Use within **6 months** after opening. The printed use-by date applies to the unopened bottle. The solution is non-sterile and contains no preservatives.

Store the other parts dry, at room temperature, protected from direct sunlight.

7. Materials

Part		Material
Head cap		<offen>
Electrodes, contact surface		Silver/silver chloride (Ag/AgCl)
Electrodes, body		<offen>
Electrodes, stud		<offen>
Felt pads		<offen>
Electrolyte solution		Sodium chloride 9 g/l in water, no additives

If you know you are allergic to any of these materials, do not use the part concerned.

8. Disposal

All parts can be disposed of with household waste. The empty glass bottle belongs in glass recycling; remaining solution can be rinsed away with water. The sensor itself is an electrical device and must not be disposed of with household waste — see its own manual.

9. Manufacturer and enquiries

Mindfield Biosystems Ltd.
Hindenburgring 4, 48599 Gronau, Germany
support@mindfield.de

Please quote the batch number (LOT) from the label of the part concerned.

Reporting incidents. If any harm to health occurs in connection with the use of this product, please report it to the address above. Serious incidents may also be reported to the German competent authority: Bundesinstitut für Arzneimittel und Medizinprodukte (BfArM), Kurt-Georg-Kiesinger-Allee 3, 53175 Bonn, Germany.

Revision of these instructions: <offen>, revision <offen>